

Vented SEVA Convenience Kit

Vented SEVA is a closed hazardous drug system enclosing:


- Contaminated vial exterior
- Liquid leakage and spills
- Vapor emissions

Using SEVA is easy:

1. Pull tether to open alcohol swab
2. Enclose vial and swab septum
3. Spike vial
4. Prepare and draw drug

Features

- Transparent 4 mil medical grade polyethylene bag
- Integral tethered alcohol swab
- Adhesive bag and vial securement
- Enclosed vented vial adapter
- Closed male luer connector compatibility
- Hand-held or hung for hands-free use
- Compatible with multiple vial sizes

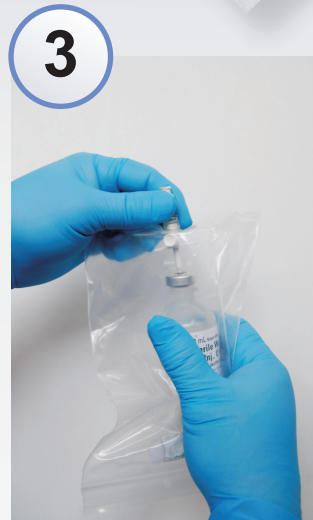

**Questions? Call or visit IntraVena at:
1-801-510-8733 • www.intravenaonline.com**

Reference Number	Description	Case Quantity
0105-001	Vented SEVA Convenience Kit, 5x8 Inch Bag	100
0105-006	Vented SEVA with Tethered Alcohol Swab Convenience Kit, 5x8 Inch Bag	100
0105-002	Vented SEVA with Tethered Alcohol Swab Convenience Kit & Texium [®] 5x8 Inch Bag	100
0106-001	SEVA Direct with Texium [®] with Tethered Alcohol Swab Convenience Kit, 5x8 Inch Bag	100
0106-007	SEVA Direct Basic with Tethered Alcohol Swab Convenience Kit, 5x8 Inch Bag	100
0107-001	Mini SEVA with Tethered Alcohol Swab Convenience Kit, 2.5x6 Inch Bag	100
0107-002	Mini SEVA with Tethered Alcohol Swab Convenience Kit & Texium [®] , 2.5x6 Inch Bag	100